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INDICATIONS

Indications

MATCH composites are used for direct reconstruction of enamel and dentin in permanent teeth and tissues for restorations of all classes I, II, III, IV, V, and VI in aesthetic corrections, laminated veneers, inlay/onlay techniques, crowns, table tops, and stabilization of anterior and posterior teeth.

Specifications and Technical Characteristics of the Product

The MATCH material family consists of MATCH 3D MASTER and MATCH CLASSIC composites, precisely following the shades of the VITA 3D MASTER and VITA CLASSICAL shade guides.

It is a nanohybrid, light-curing, radiopaque restorative composite with zirconia filler, used in direct and indirect restoration procedures in both anterior and posterior regions. It is indicated for restorations of all classes. MATCH composite is highly aesthetic, providing translucency, fluorescence, and opalescence. The enamels are manufactured to match exactly the colors of the VITA 3D MASTER and VITA CLASSICAL shade guides. MATCH S Dentins simulate the dentin substrate with different opacities, providing the base for the restoration. MATCH CLEAR and MATCH CLEAR BLUE composites for body and incisal areas complement the MATCH system. MATCH FLOW composite, with high opalescence, allows for greater naturalness in restorations. The MATCH BOND U Universal Adhesive is alcohol- and acetone-free, recommended for application on enamel and dentin after proper acid conditioning. The materials polymerize (harden) using halogen or LED light.

USE INSTRUCTIONS

1. Perform a prophylaxis on the tooth to be treated and adjacent teeth using a Robinson brush and a fluoride-free prophylactic paste. 2. Select the appropriate shade of restorative material following the VITA 3D MASTER or VITA Classical shade guide. The MATCH composite colors match those of the VITA 3D MASTER or VITA Classical shade guides. 3. Ensure absolute isolation. 4. Prepare the cavity according to the recommendations for composite restorations. 5. Perform pulpal protection when necessary using a liner material such as calcium hydroxide or glass ionomer, according to your clinical assessment. 6. To anatomically restore the teeth and minimize excess of material removal, use common or preformed polyester matrices. For posterior teeth, polyester matrices should be used. Pre-wedging will facilitate the achievement of proper proximal contacts. 7. Perform the acid etching technique following the instructions for the etching product being used. 8. 8. Apply MATCH BOND adhesive or a similar product according to the instructions for use. 9. Place the required amount of MATCH composite inside the cavity preparation, following incremental layering techniques with an appropriate instrument, firmly adapting it to the cavity walls. Each applied layer should not exceed 2mm in thickness. 10. Light Cure Each Layer for up to 40 seconds. After removing the matrix, the composite should undergo final additional polymerization. Contour the restoration surface with fine diamond burs or stones for finishing. Check occlusion using articulating

paper. Carefully adjust the occlusion by removing excess material with silicone tips, polishing discs, or finishing strips..

PRODUCT PRESENTATIONS

MATCH RESIN: available in MATCH 3D, MATCH CLASSIC, MATCH DENTIN, MATCH CLEAR, MATCH CLEAR BLUE, MATCH FLOW, MATCH BOND U, and MATCH BOND E.D.

MATCH CLEAR: CLEAR. MATCH CLEAR BLUE: CLEAR BLUE. MATCH FLOW: A1/A2/A3. MATCH DENTIN: D1/D2/D3/D4. MATCH BOND U: UNIVERSAL ADHESIVE

PRECAUTIONS

- 1 - This product is manufactured to be used only as specified in the instructions for use. Any use other than as described in the instructions is the sole responsibility of the professional using it.
- 2 - This composite is light-sensitive. Therefore, excessively intense ambient lighting may cause premature polymerization. Keep the product tightly closed when not in use. After being dispensed from the syringe, it should be used immediately.
- 3 - Keep out of reach of children.
- 4 - Should not be used with products containing phenol (phenol, clove oil, copal varnish) or zinc oxide cements with eugenol, as they prevent or affect polymerization.
- 5 - Discoloration may occur when using cationic mouthwashes, as well as plaque developers and chlorhexidine.
- 6 - After use, the material should be properly sealed.
- 7 - All dental materials should be used exclusively by dentists and authorized dental laboratories.

RESTRICTIONS OR CONTRAINDICATIONS

- 1 - As a preventive measure to avoid a possible pulpal reaction, use a calcium hydroxide cement, glass ionomer cement, or another type of cavity liner before inserting the material into the cavity.
- 2 - Product Clarifications 2.1 - Metal instruments may be used. 2.2 - During handling, avoid exposing the material to direct or intense ambient lighting.
- 3 - Storage and Transport: 3.1 - The product must remain in its original sealed packaging after use. 3.2 - Store in a cool, dry place at room temperature and protected from direct light. 3.3 - Do not use after the expiration date. The expiration date on the packaging represents the maximum usage period for the product.
- 4 - Shelf Life: 36 months from the date of manufacture. 5 - Storage Conditions: 5.1 - Store at temperatures between 5°C and 30°C, keeping the product in its original, sealed packaging. 5.2 - Protect the product from direct sunlight exposure. 5.3 - Do not freeze the product. 5.4 - If stored in a refrigerator, wait approximately 15 minutes before use. 5.5 - The ideal usage temperature range is between 20°C and 25°C.

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RESTRICTIONS O CONTRAINDICATIONES

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