MASTER COLOR

RESINA COMPOSTA FOTOPOLIMERIZÁVEL

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LIGHT-CURABLE COMPOSITE RESIN MASTER COLOR

1. PRODUCT IDENTIFICATION

Trade Name: Master Color.

Pharmaceutical Form: Light-Curable Composite Resin.

Presentation: Syringes of 1g, 1.2g, 2.0g, 2.5g and 3.0g.

Usage: Exclusive for dental professionals.

2. COMPOSITION

Components: Monomers – Photoinitiators – Fillers - Pigments

3. APRESENTATION

MASTERCOLOR characterization pigments

Yellow – Blue – White – Caramel – Gray – Mustard – Black – Pink – Red – Violet

MASTERCOLOR gingiva pigments

Light pink – Midium pink – Brown pink – Black pink – Purple ponk – Red pink

Colorless Base (Master Base): Used for pigment dilution or the creation of new shades.

4. INDICATIONS

Master Color is indicated for the extrinsic characterization of:

Composite resin teeth (ceromer), 3D-printed resin dental prostheses, acrylic teeth, artificial gingiva. The product is used for inlays, onlays, veneers and other resin-based dental prostheses.

5. INSTRUCTIONS FOR USE

5.1. Application: the pigments can be used individually or mixed with MasterBase. The containers must be completely clean and free of residues. Apply the pure or mixed material gently onto the piece.

5.2. Post-Curing: perform post-curing in a UV chamber for 1 to 2 minutes after each layer. The final layer should be cured for 5 to 10 minutes, depending on the equipment used.

For better aesthetic results and surface protection, it is recommended to apply a layer of the MasterSeal light-curable varnish at the end of the characterization process, after the post-curing step. Follow the specific instructions for the MasterSeal product to ensure proper application and polymerization.

5.3. Polymerization: use UV LED polymerization devices (wavelength: 360-500 nm, maximum of 470 nm). Clinical light-curing equipment for direct restorations should be avoided.

6. WARNINGS AND PRECAUTIONS

Product intended for use exclusively by dentists and dental technicians. The use of Personal Protective Equipment (PPE) is mandatory when handling the product.

The resin is light-sensitive and must be kept sealed and protected from direct light exposure. Use is contraindicated for patients with known allergies to the components.

7. STORAGE AND SHELF LIFE

Storage Conditions: keep in a dry place at room temperature (15-30°C), away from light and heat.

Shelf Life: 36 months from the manufacturing date when properly stored. If refrigerated, allow the product to return to room temperature before use.

8. DISPOSAL

Syringes must be disposed of after use in accordance with local regulations for dental waste.

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