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## DENTAL SUTURE NEEDLE WITH SILK THREAD

**Indication:** Indicated for suture of gingival surfaces after surgical procedures.

Exclusive use for healthcare professionals.

**Composition:** Stainless steel 304 needle and braided silk thread.

Valid date: 5 years after manufacture.

**Contraindications:** None

**Instructions for use:** After the surgical procedure, proceed the suture according to the technique of choice of the professional.

**Warning:** Single use Destroy after use. Do not use if pack is damaged.

**Storage:** Store at room temperature.

**Working principle:**

The intraoral use of the suture thread must be conducted by dentists or personnel with proper training. The professional handles the product with dental pliers through puncture with the needle and insertion of the wire in the gingival area and subsequent cut and disposal of the needle with the thread. The materials are non-toxic, non-absorbable and non-impregnated. Its use is short-term, remaining on the gums for up to 30 days. Comes with the surgical numbers 2.0, 3.0, 4.0, 5.0 and 6.0 and can be 45 cm or 75 cm of length. The needle is made of stainless steel triangular or circular shape, with angles of 180° (1/2 circle) or 135° (3/8 of a circle).

These have lengths of 1.5 cm, 1.7 cm or 2.0 cm

Sterile product -Sterilized by ethylene oxide.

Exclusive use for healthcare professionals.

Single-use product.