# RESILAB 3D

## SAND PREMIUM MODEL

405 nm DLP/LCD

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### UV RESIN FOR 3D PRINTING

**CONTAINS:** UV-curable resin (UVA/UVB), exclusively for use in 3D printers.

**INDICATIONS:** Dental models, models for aligners, and dies.

**INSTRUCTIONS FOR USE:**

Shake the resin well for at least 2 minutes or use a mixer for a minimum of 5 minutes.

Pour the required amount into the printer's tank to begin printing.

After printing, spray isopropyl alcohol on the printed object and use a brush to remove excess resin.

Alternatively, use the Light & Clean washer for cleaning, ensuring that the total immersion time does not exceed 3 minutes.

Dry the piece thoroughly to remove all excess material.

Place the piece in a UV curing chamber (405 nm power) for 8 minutes.

**NOTE:** This time may be adjusted depending on the power of your UV chamber. After printing, the resin should be removed from the tank, filtered, and stored in a UV-protected container. Always handle this product using gloves, a mask, and eye protection.

**PRECAUTIONS:**

Store in a dry, well-ventilated place, away from heat sources and direct light.

Keep the container tightly closed when not in use.

Avoid contact with skin, eyes, and clothing, as it may cause sensitization through skin contact.

Do not ingest or inhale.

Use gloves and eye protection when handling the product.

**KEEP OUT OF REACH OF CHILDREN AND PETS.**

**FIRST AID MEASURES:**

**SKIN CONTACT:** Wipe off with absorbent paper or a towel. Immediately wash with plenty of water and soap. Do not use organic solvents. In case of dermatitis, seek medical assistance.

**EYE CONTACT:** Rinse thoroughly with running water for 20 to 30 minutes.

**INHALATION:** Move to a fresh, well-ventilated area in case of strong inhalation.

**INGESTION:** Do not induce vomiting. The affected person should drink 0.5 to 0.8 liters of water and seek medical assistance.

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