MasterSeal

Glaze fotopolimerizável

WILCOS

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Use indication:

MasterSeal is a light curing glaze. Thus, once applied, the varnish penetrates the porosities of the acrylic piece and, after being subjected to polymerization, seals them.

This seal stabilizes and gives smoothness to the surface of the resin, making the piece resistant to abrasion and providing high gloss.

How to use:

- Gently apply a thin, even layer, using a soft, point brush, avoiding the formation of bubbles.

- Let it act for 20 seconds before polymerizing.

- Polymerize, using the polymerization time as described below:

a) U.V.A. and Halogens, polymerize for 5 minutes.

b) Strobo lamp fixtures such as UNIX-S and POWER LUX, polymerize for 3 minutes.

Precautions:

The product is manufactured to be used only as stated in the instructions for use. The use in a way different from the instructions for use is the sole and exclusive responsibility of the professional who uses it.

The product is for professional dental use.

Always close the bottle well after use to avoid evaporation of the components and a consequent change in their characteristics.

Handle using a mask, gloves and goggles for proper protection, thus avoiding inhalation and contact with the skin and eyes.

Adverse effects:

Do not use in works for people with hypersensitivity to the product components.

Adverse effects:

- Contact with the eyes: May cause irritation to the eyes.
- Skin Contact: May cause skin irritation.
- Inhalation: Irritating the respiratory system, may cause headaches.

Storage

It must remain in its original, unopened packaging after use. protect from direct light.

Waste disposal and packaging in common trash.

Warnings

Contains methyl methacrylate which may cause an allergic reaction in susceptible individuals.

MasterSeal is a flammable liquid, keep away from fire or heat sources.

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