# MasterSeal

## Glaze fotopolimerizável

# WILCOS

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### Use indication:

MasterSeal is a light curing glaze. Thus, once applied, the varnish penetrates the porosities of the acrylic piece and, after being subjected to polymerization, seals them.

This seal stabilizes and gives smoothness to the surface of the resin, making the piece resistant to abrasion and providing high gloss.

### How to use:

To open, turn the cap counterclockwise.

Remove the dropper insert from the packaging. Then, open the bottle and remove the sealing insert from the neck of the bottle. After removal, firmly fit the dropper insert into the opening of the bottle.

Note: The dropper insert must be fully seated in the bottle; otherwise, it will not be possible to close it properly.

- Gently apply a thin, even layer, using a soft point brush, avoiding the formation of bubbles.

- Let it act for 20 seconds before polymerizing.

- Polymerize, using the polymerization time as described below:

### **Precautions:**

The product is manufactured to be used only as stated in the instructions for use. The use in a different way from the instructions for use is of sole and exclusive responsibility of the professional who uses it.

The product is for professional dental use.

Always close the bottle well after use to avoid evaporation of the components and a consequent change in their characteristics.

Handle using a mask, gloves and goggles for proper protection, thus avoiding inhalation and contact with the skin and eyes.

### **Adverse effects:**

Do not use in works for people with hypersensitivity to the product components.

- Contact with the eyes: May cause irritation to the eyes.

- Skin Contact: May cause skin irritation.

- Inhalation: In case of inhalation may cause headaches.

Storage:

It must remain in its original unopened packaging after use. Protect from direct light.

Waste and package disposal in common trash.

### **Special instructions:**

Indicated for use in total or partial prosthodontics with acrylic resins, 3D printed resin materials and composites. Also to seal porosities, it offers high shine to acrylic material.

### **Warnings**

Contains methyl methacrylate which may cause an allergic reaction in susceptible individuals.

MasterSeal is a flammable liquid, keep away from fire or heat sources.

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